

Face Shield Assembly Instructions

Prusa RC3 Design



Made in the UK

In this kit:



Before unpacking the pieces:

Wash your hands thoroughly.

If you have PPE, wear it to unpack and assemble this unit.

Important

Face shield designed for face and eye protection against droplets and splashes of liquids.

This face shield is meant for single use.

Not suitable for steam sterilisation or any method that involves high temperatures such as high autoclave.

We advise to wait 72 hours after packing date until usage.

1. Attach the visor (A) to the headband (B)



1. Hold the headband with the row of nubbins at the bottom.



2. Insert the two middle holes of the visor through the two centre nubbins on the headband.



3. Insert the two remaining nubbins of the headband into the remaining holes on the right and left-hand side of the visor. You may need to bend the headband outwards slightly, and pull on the visor to reach the nubbins with the holes.

2. Attach the bottom reinforcement (C)



1. Insert the visor into the bottom reinforcement, checking that both pieces are centred.

3. Attach the elastic strap (D)

If you received a Buttonhole elastic



1. Insert the lug of the headband through the last buttonhole of the elastic. Use either of the elastic ends.
2. Repeat on the lug on the opposite side, adjusting to suit the size of your head.

If you received a Plain elastic



1. Wrap one extreme of the elastic around one of the lugs at the back of the headband.



2. Tie a knot and pull tight.
3. Repeat on the lug on the opposite side, adjusting to suit the size of your head.

Latest information and assembly

Watch the video at:

<https://prusa3d.com/covid19>



Face Shield Kit Regulatory Information

Prusa RC3 Design



Made in the UK

Product & use

Contains



Single Use



Contains Natural Rubber Product

Face shield designed for face and eye protection against droplets and splashes of liquids.

We advise to wait 72 hours after packing date until usage.

Supplied by 3DCrowd
www.3dcrowd.org.uk

Use

- Recipients of the face shields are responsible for using their own judgement before deciding to order, receive, distribute and use these face shields in any setting.
- This face shield can play a part in reducing risk to users, but they are responsible for following established government and their institutional guidance as to how to protect themselves from COVID-19.
- These face shields are single use only. Discard after use. Do not reuse or re-sterilise. It is

expected that these face shields will be used in conjunction with a face mask protecting the nose and mouth, in accordance with relevant guidelines. Face shields are not a replacement for or negate the need for users to stringently follow the recommended hygiene practices or other relevant guidelines.

- Recipients of these face shields are required to provide a copy of this Disclaimer to any third party before distributing them further.

Get involved

Help Us Help You

Thanks to the generous donations received of £2 per shield we were able to send you these Face Shields.

Support us by donating at www.3dcrowd.fund



Contact Us

For queries and information, email us at orders@3dcrowd.org.uk

Feedback

Let us know about your experience

<https://www.surveymonkey.co.uk/r/7XZSB9H>



You have urgent work to do, but this information is important

Disclaimer

In response to the COVID-19 pandemic, a network of volunteers have combined efforts to produce face shields using a network of 3D printers and coordinate the distribution of them to frontline medical staff.

Production & Certification

These face shields are made to a common design but are made out of various combinations of materials. The 3DCrowd RC3 Face Shield is undergoing an accelerated process put in place by the EU and the UK government to allow testing and certification of PPE for Healthcare workers as Class II PPE for healthcare under BS EN 166:2002 as a matter of urgency.

Per COMMISSION RECOMMENDATION (EU) 2020/403, it is legal for 3DCrowd to supply these to healthcare workers during the Covid-19 emergency while certification is pending.

Limitation of Warranty

No representation, promise, express warranty or implied warranty concerning the suitability of these shields for any medical or personal protection use including, without limitation, their effectiveness to filter, block or protect against any pathogen or particle, including COVID-19.

3DCrowd CIC registered in England with number: 12535979